Read Instruction Manual **1** and **2** before use.

1. Intended use

This device is a digital monitor operating on the oscillometric method, intended for use in measuring blood pressure and pulse rate of adult patients who have an understanding of this instruction manual with the arm circumference range printed on the arm cuff. The device detects the appearance of irregular heartbeats during measurement and shows warning symbols with the measurement result (except Model IM2 Basic). It is mainly designed for general household use.

Please read section "2. Important Safety Information" before using the device.

2. Important Safety Information

▲ 2.1 This warning symbol ▲ indicates a potentially hazardous situation which, if not avoided, could result in death or serious injury.

General Usage

- ▲ 2.1.1 Consult your physician before using the device for any of the following conditions: common arrhythmias such as atrial or ventricular premature beats, atrial fibrillation, arterial sclerosis, poor perfusion, diabetes, age, pregnancy, pre-eclampsia, renal diseases. Note that motion, trembling, shivering may affect the measurement reading.
- ▲ 2.1.2 Do not adjust medications based on readings from this blood pressure monitor. Take medications as prescribed by your physician.
- Only a physician is qualified to diagnose and treat high blood pressure.
- ▲ 2.1.3 Do not use the device on an injured arm or an arm under medical treatment.
- ▲ 2.1.4 Do not apply the arm cuff to the arm while on an intravenous drip or blood transfusion.
- ▲ 2.1.5 Consult your physician before using the device on an arm with an arterio-venous (A-V) shunt.
- ▲ 2.1.6 Do not use the device with other medical electrical (ME) equipment simultaneously. This may result in incorrect operation of the device and/or cause an inaccurate reading.
- ▲ 2.1.7 Do not use the device near high frequency (HF) surgical equipment, MRI, or CT scanners, or in an oxygen rich environment. This may result in incorrect operation of the device and/or cause an inaccurate reading.
- ▲ 2.1.8 The air tube or AC adapter cable may cause accidental strangulation to young children and infants.
- ▲ 2.1.9 The device contains small parts that may cause a choking hazard if swallowed by young children and infants.
- ▲ 2.1.10 Stop using the device and consult your physician if you experience skin irritation or other troubles.

AC Adapter (optional) Usage

- ▲ 2.2.1 Fully insert the power plug into the outlet.
- ▲ 2.2.15 When disconnecting the power plug from the outlet, do not pull the power cord. Remove by holding the plug.
- ▲ 2.2.16 When handling the power cord, take care not to damage or mishandle it in any way.
- ▲ 2.2.17 Keep the power plug free from dust.
- ▲ 2.2.18 Unplug the monitor when not in use.
- ▲ 2.2.19 Disconnect the power plug before cleaning.
- ▲ 2.2.20 Use only the original adapter designed for this device. Use of unsupported adapters may damage and/or may be hazardous to the device.

Battery Usage

- ▲ 2.2.21 Ensure the polarity is correctly aligned when inserting batteries.
- ▲ 2.2.22 Use only 4 "AA" alkaline or manganese batteries with this device. Do not use other types of batteries. Do not use new and used batteries together.
- ▲ 2.2.23 Remove the batteries if the device will not be used for 3 months or longer.
- ▲ 2.2.24 Do not use batteries that are outside their expiry date.

2.3 General Precautions

- 2.3.1 Do not create the arm cuff or the air tube excessively.
- 2.3.2 Take care not to compress the air tube while taking a measurement. This may cause injuries by interrupting blood flow.
- 2.3.3 When you take a measurement on the right arm, the air tube should be at the side of your elbow. Be careful not to rest your arm on the air tube.

- ▲ 2.3.4 Blood pressure may differ between the right and left arm, and may result in a different measurement value. Please always use the same arm for measurements. If the values between both arms differ substantially, please check with your physician on which arm to use for your measurements.
- 2.3.5 To unplug the air plug, pull on the air plug at the connection with the monitor, and not the tube itself.
- 2.3.6 Do not drop the monitor or subject the device to strong shock or vibrations.
- 2.3.7 Do not wash/clean the device using water, petrol, thinners or similar solvents, abrasive or volatile cleaners.
- 2.3.8 Do not use/store the device outside the specified environment. It may lead to inaccurate measurements.
- 2.3.9 Do not inflate the arm cuff when it is not wrapped around your arm.

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***In the clinical validation study, the 5th phase was used on 85 subjects for determination of diastolic blood pressure**

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The high blood pressure definition is based on the JNC7 standard.

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****EC standard: Non-invasive sphygmanometers Part 1: General requirements and Part 3: Supplementary requirements for electromechanical blood pressure measuring systems**

- FR** Norme CE: sphygmanomètres non invasifs Partie 1: exigences générales et Partie 3: exigences supplémentaires pour les systèmes électromécaniques de mesure de la tension artérielle
- DE** EU-Norm: Nicht-invasive Blutdruckmessgeräte Teil 1: Allgemeine Anforderungen und Teil 3: Ergänzende Anforderungen für elektromechanische Blutdruckmesssysteme
- NL** Sfygmometri norm: Invasieve Part 1: Requisiti supplementari per sistemi elettromeccanici di misurazione della pressione sanguigna
- ES** Norma CE: Sfigmomanómetros no invasivos Parte 1: Requisitos generales y Parte 3: Requisitos complementarios para los sistemas electromecánicos de medición de la presión arterial
- RU** ЕC-номм: Нет-инвазивные блудотмерные Deel 1: Algemene eisen en Deel 3: Aanvullende eisen voor elektromechanische bloeddrukmeetstelsels
- TR** Standart EC: "Неинвазивные сфигмоманометры", часть 1 "Общие требования" и часть 3 "Дополнительные требования для электромеханических систем измерения артериального давления"
- AR** EC standard: Non-invasif sfigmomanometreler Kısmı 1: Genel gereksinimler ve Kısmı 3: Elektromekanik kan basıncı ölçme sistemleri için ek gereksinimler
- FR** **EC** **AR** المقاييس ضغط الدم غير الباضعة الجزء 1: المتطلبات التكميلية لأجهزة الباضعة الجزء 3: المتطلبات التكميلية لأجهزة الباضعة

This device has not been validated for use during pregnancy

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Specifications mentioned in this manual are subject to change without notice.

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Calibration and service: The accuracy of this device has been carefully tested and is designed for a long service life. It is generally recommended to have the device inspected every 2 years to ensure correct functioning and accuracy. Please consult your Omron retail outlet or distributor.

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