# beurer

# MG 17

<b>DE</b> Spa-Mini Massager Gebrauchsanweisung.....2	<b>IT</b> Apparecchio mini per massaggi Spa Istruzioni per l'uso .....53
<b>EN</b> Spa mini massager Instruction for Use..... 15	<b>TR</b> Spa Mini masaj aleti Kullanma Talimatı.....65
<b>FR</b> Mini appareil de massage spa Mode d'emploi .....27	<b>RU</b> Мини-массажер СПА Инструкция по применению .....77
<b>ES</b> Aparato de masaje Spa-Mini Instrucciones para el uso.....40	<b>PL</b> Miniurządzenie do masażu spa Instrukcja obsługi.....89

# DEUTSCH

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## ENGLISH

**Read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.**

### **WARNING**

- The device is intended only for domestic/private use, not for commercial use.
- This device can be used by children over the age of 8 and by people with reduced physical, sensory or mental skills or a lack of experience or knowledge, provided that they are supervised or have been instructed on how to use the device safely and are aware of the consequent risks of use.

- Children must not play with the device.
- Under no circumstances should you open or repair the device yourself, as faultless functionality can no longer be guaranteed thereafter. Failure to comply will invalidate the warranty.
- Cleaning and user maintenance must not be performed by children unless supervised.

**Dear customer,**

Thank you for choosing a product from our range. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage, beauty, baby and air.

With kind regards,  
Your Beurer team

## Table of contents

1. Included in delivery.....	17
2. Signs and symbols .....	17
3. Warnings and safety notes.....	19
4. Intended use.....	20
5. Initial use.....	22
6. Operation.....	23
7. Cleaning and maintenance.....	23
8. Disposal.....	24
9. Technical specifications .....	24
10. Warranty/service.....	25

## 1. Included in delivery

- 1 x Mini massager
- 2 x Attachments (1 x loofah, 1 x cotton)
- 2 x AAA batteries

## 2. Signs and symbols

The following symbols appear in the instructions for use and on the device:

	<b>WARNING</b>	Warning notice indicating a risk of injury or damage to health.
	<b>IMPORTANT</b>	Safety note indicating possible damage to the device/accessory.

	<b>Note</b>	Note on important information.
	Read the instructions for use	
	<b>CE labelling</b> This product satisfies the requirements of the applicable European and national directives.	
	Disposal in accordance with the Waste Electrical and Electronic Equipment EC Directive – WEEE	
	Manufacturer	
	Dispose of packaging in an environmentally friendly manner	
	The products demonstrably meet the requirements of the Technical Regulations of the EAEU.	
	Direct current	
	Suitable for use in a bath or shower.	

### 3. Warnings and safety notes

Read these instructions for use carefully! Non-observance of the following information may result in personal injury or material damage. Store these instructions for use and make them accessible to other users. Make sure you include these instructions for use when handing over the device to third parties.

#### Important

- Before use, ensure that the device and accessories display no visible damage. If you have any doubts, do not use the device and contact your retailer or the specified Customer Service address.
- Never open the housing.
- Keep the device away from pointed or sharp objects.
- If the device has been dropped or has suffered any other damage, it must no longer be used.
- The exfoliation attachment can be reordered using order number 646.39.

#### WARNING

- Keep packaging material away from children. There is the risk of suffocation.
- Children must be supervised at all times whilst using the device.

#### Notes on handling batteries

- If your skin or eyes come into contact with battery fluid, rinse the affected area with water and seek medical assistance.
-  **Choking hazard!** Small children may swallow and choke on batteries. Therefore, store batteries out of the reach of small children.

- Observe the plus (+) and minus (-) polarity signs.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- Protect batteries from excessive heat.
-  **Risk of explosion!** Do not throw batteries into a fire.
- Do not charge or short-circuit batteries.
- If the device is not to be used for a relatively long period, take the batteries out of the battery compartment.
- Use identical or equivalent battery types only.
- Always replace all batteries at the same time.
- Do not use rechargeable batteries.
- Do not disassemble, open or crush the batteries.

## Repairs

### **WARNING**

- Only specialist personnel may perform repairs on electrical devices. Improper repairs may subject users to considerable danger. For repairs, please contact Customer Services or an authorised retailer.

## 4. Intended use

- The device is only intended for the purpose described in these instructions for use. The manufacturer is not liable for damage resulting from improper or careless use.

- The device is intended to massage individual parts of the human body. If you have a pacemaker, implant or other aids, or if you are not sure that the device is suitable for you, consult your doctor prior to use.

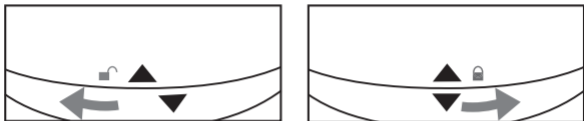
## **WARNING**

Do not use the massager:

- On animals,
- If you suffer from a medical abnormality or injury in the area to be massaged (e.g. herniated disc, open wounds),
- Under any circumstances for massages in the area around the heart if you wear a pacemaker. If this is the case, consult your doctor regarding other parts of the body first,
- On swollen, burnt, inflamed or damaged skin and body parts of any kind,
- On cuts, capillaries, varicose veins, acne, couperose, herpes or other skin diseases,
- During pregnancy,
- On the face (eyes), on the larynx or other particularly sensitive body parts,
- While sleeping,
- After the consumption of medicines or alcohol (reduced cognition!),
- In a vehicle,
- In the pubic area.


## 5. Initial use

- Remove the packaging.
- Check the device and batteries for damage.

Turn the battery compartment lid. Insert 2 AAA batteries in accordance with the marking. Reapply the battery compartment lid, ensuring correct alignment.

## 6. Operation

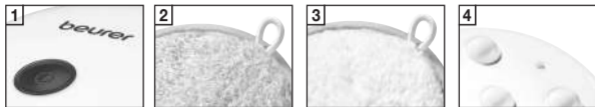At the touch of a button **1** (1x on  / 2x  off)

**Secure the attachment and choose the desired side of the spa attachment:**

**2** Loofah: exfoliation side

**3** Cotton: cleaning side

Secure the loops of the spa attachment at the 4 straps on the device.

**4** 4 openings on the base of the device ensure potential water drainage.

## 7. Cleaning and maintenance

- Clean the body brush using only the methods specified.
- Clean the body brush after use with a soft, slightly damp cloth. If it is very dirty, you can also moisten the cloth with a mild soapy solution. Do not use any abrasive cleaning products.
- Protect the body brush from knocks, moisture, dust, chemicals, drastic changes in temperature and nearby sources of heat (ovens, heaters).

## 8. Disposal

### IMPORTANT

- Batteries must not be disposed of in the household waste. They may contain poisonous heavy metals and are subject to special refuse treatment.
- The codes below are printed on batteries containing harmful substances  
Pb = Battery contains lead  
Cd = Battery contains cadmium  
Hg = Battery contains mercury

For environmental reasons, do not dispose of the device in the household waste at the end of its service life.

Dispose of the device at a suitable local collection or recycling point in your country.

Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment).

If you have any questions, please contact the local authorities responsible for waste disposal.

## 9. Technical specifications

Weight	Approx. 222 g
Battery	2 x 1.5 V  Batterie AAA LR03
Device dimensions	8.5 x 10.0 x 8.0 cm

## 10. Warranty/service

Beurer GmbH, Söflinger Straße 218, 89077 Ulm, Germany (hereinafter referred to as “Beurer”) provides a warranty for this product, subject to the requirements below and to the extent described as follows.

**The warranty conditions below shall not affect the seller’s statutory warranty obligations which ensue from the sales agreement with the buyer. The warranty shall apply without prejudice to any mandatory statutory provisions on liability.**

Beurer guarantees the perfect functionality and completeness of this product.

The worldwide warranty period is 3 years, commencing from the purchase of the new, unused product from the seller.

The warranty only applies to products purchased by the buyer as a consumer and used exclusively for personal purposes in the context of domestic use. German law shall apply.

During the warranty period, should this product prove to be incomplete or defective in functionality in accordance with the following provisions, Beurer shall carry out a repair or a replacement delivery free of charge, in accordance with these warranty conditions.

**If the buyer wishes to make a warranty claim, they should approach their local retailer in the first instance: see the attached “International Service” list of service addresses.**

The buyer will then receive further information about the processing of the warranty claim, e.g. where they can send the product and what documentation is required.

A warranty claim shall only be considered if the buyer can provide Beurer, or an authorised Beurer partner, with

- a copy of the invoice/purchase receipt, and
- the original product.

The following are explicitly excluded from this warranty:

- deterioration due to normal use or consumption of the product;
- accessories supplied with this product which are worn out or used up through proper use (e.g. batteries, rechargeable batteries, cuffs, seals, electrodes, light sources, attachments and nebuliser accessories);
- products that are used, cleaned, stored or maintained improperly and/or contrary to the provisions of the instructions for use, as well as products that have been opened, repaired or modified by the buyer or by a service centre not authorised by Beurer;
- damage that arises during transport between manufacturer and customer, or between service centre and customer;
- products purchased as seconds or as used goods;
- consequential damage arising from a fault in this product (however, in this case, claims may exist arising from product liability or other compulsory statutory liability provisions).

Repairs or an exchange in full do not extend the warranty period under any circumstances.

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