


Compressor Nebuliser

NEB 200

NEB 200 Ni-V9-0521 - Revision Date: 2021-10-22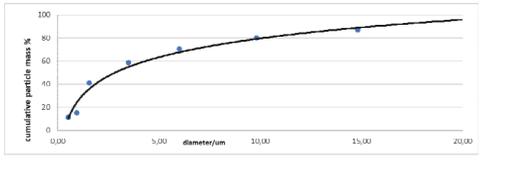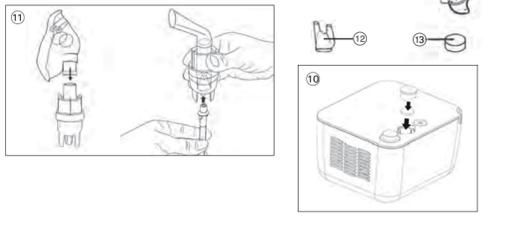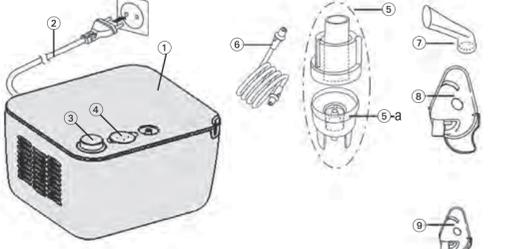

### Guarantee Card

Name of Purchaser / Inköpares namn / Ostajan nimi / Forhandlers navn / Kopers navn / Pirçida vards / Pirkeçivisi / Ostaja imi / Ф.И.О. покупателя

Serial Number / Serienummer / Sarjanumero / Serienummer / Serienummer / Sériás numéri / Serijs number / Serianumber / Серийный номер

Date of Purchase / Inköpsdatum / Ostopäivämäärä / Katsdato / Kijpsdato / Iegados datums / Parvadimo data / Ostukuparje / Дата покупки

Specialist Dealer / Återförsäljare / Alank kauppias / Special-forhandler / Specialist forhandler / Specialists -pärstävits / Paradvusi istajai / Ametik müügiesindaja / Специализированный дилер

Globalcare Medical Technology Co., Ltd  
7th Building, 39 Middle Industrial Main Road,  
European Industrial Zone, Xixian Town,  
528415 Zhongshan City, Guangdong Province, PRC

Donava Lifescience Consulting Srl  
Piazza Albania, 10, 00153 Roma Italy

## Compressor Nebuliser

- Piston compressor
- Power lead
- ON/OFF Switch
- Air filter compartment
- Nebuliser
  - Vaporiser head
- Air tube
- Mouthpiece
- Adult face mask
- Child face mask
- Replacing air filter
- Assembling nebuliser kit
- Noise piece
- Air filter

Dear Customer,  
This nebuliser is an aerosol therapy system suitable for domestic use. This device is used for the nebulisation of liquids and liquid medication (aerosols) and for the treatment of the upper and lower respiratory tract. If you have any questions, problems or want to order spare parts please contact your local MicroLife-Customer Service. Your dealer or pharmacy will be able to give you the address of the MicroLife dealer in your country. Alternatively, visit the internet at www.microlife.com where you will find a wealth of invaluable information on our products.

### 1. Explanation of Symbols

- Electronic devices must be disposed of in accordance with the locally applicable regulations, not with domestic waste.
- Read the instructions carefully before using this device.
- Type BF applied part
- Class II equipment
- Serial number
- Reference number
- Manufacturer
- ON
- OFF

IP21 Protection against solid foreign objects and harmful effects due to the ingress of water

CE Marking of Conformity

### 2. Important Safety Instructions

- Follow instructions for use. This document provides important product operation and safety information regarding this device. Please read this document thoroughly before using the device and keep for future reference.
- This device may only be used for the purposes described in these instructions. The manufacturer cannot be held liable for damage caused by incorrect application.
- Retain instructions in a safe place for future reference.
- Do not operate the unit in presence of any anesthetic mixture inflammable with oxygen or nitrogen protoxide.
- This device is not suitable for anaesthesia and lung ventilation.
- This device should only be used with original accessories as shown in these instructions.
- Do not use this device if you think it is damaged or notice anything unusual.
- Never open this device.
- This device comprises sensitive components and must be treated with caution. Observe the storage and operating conditions described in the «Technical Specifications» section.
- Protect it from:
  - water and moisture
  - extreme temperatures
  - impact and dropping
  - contamination and dust
  - direct sunlight
  - heat and cold
- This device compresses sensitive components and must be treated with caution. Observe the storage and operating conditions described in the «Technical Specifications» section.
- Never open this device.
- This device comprises sensitive components and must be treated with caution. Observe the storage and operating conditions described in the «Technical Specifications» section.

### 3. Replacement of the nebuliser

Replace the nebuliser (5) after a long period of inactivity, in cases where it shows deformities, breakage, or when the vapouriser head (5-a) is obstructed with dry medicine. Use only what we recommend to replace the nebuliser after a period between 6 months and 1 year depending on the usage.

### 4. Cleaning and Disinfecting

Thoroughly clean all components to remove medication residuals and possible impurities after each treatment. Use a soft and dry cloth with non-abrasive cleaners to clean the compressor.

### 5. Maintenance, Care, and Service

Order all spare parts from your dealer or pharmacist, or contact MicroLife-Service (see foreword).

### 6. Malfunctions and Actions to take

The device cannot be switched on

- Ensure the power lead (2) is correctly plugged into the socket.
- Ensure the ON/OFF switch (3) is in the position «I».
- Make sure that the device has been operating within operating limits indicated in this manual (30 min On / 30 min Off).

### 7. Guarantee

This device is covered by a 5 year guarantee from the date of purchase. During this guarantee period, at our discretion, MicroLife will repair or replace the defective product free of charge.

Opening or altering the device invalidates the guarantee. The following items are excluded from the guarantee:

- Transport costs and risks of transport.
- Damage caused by incorrect application or non-compliance with the instructions for use.
- Damage caused by use of adapters, simple or multiple, indicated on adapters and extension cables.
- Damage caused by accident or misuse.
- Packaging/storage material and instructions for use.
- Regular checks and maintenance (calibration).
- Accessories and wearing parts: Nebuliser, masks, mouthpiece, noise piece, tube, filters, nasal washer (optional).

Should guarantee service be required, please contact the dealer from where the product was purchased, or your local MicroLife service. You may contact your local MicroLife service through our website: www.microlife.com/support

Compensation is limited to the value of the product. The guarantee will not be granted if the complete product is returned with the original invoice. Repair or replacement within guarantee does not prolong or renew the guarantee period. The legal claims and rights of consumers are not limited by this guarantee.

### 8. Technical Specifications

**Nebulisation rate:** 0.4 ml/min. (NaCl 0.9%)  
**Particle size:** 63% < 5 µm  
2.83 µm (MMAD)  
**Max. free air flow:** 15 l/min.  
**Operating air flow:** 5.31 l/min.  
**Acoustic noise level:** 52 dBA  
**Power source:** 230V 50 Hz AC  
**Power lead length:** 1.6 m  
**Residual volume:** min. 2 ml; max. 8 ml  
**Operating limits:** 30 min. On / 30 min. Off  
**Operating conditions:** 10 - 40 °C / -13 - +158 °F  
30 - 85 % relative maximum humidity  
700 - 1060 hPa Atmospheric pressure

**Storage and shipping conditions:** 20 - +60 °C / -4 - +140 °F  
20 - 95 % relative maximum humidity  
700 - 1060 hPa Atmospheric pressure

**Weight:** 1306 g  
**Dimensions:** 103 x 160 x 140 mm  
**IP Class:** IP21  
**Reference to standards:** EN 13544-1;EN60601-1; EN60601-2; EN60601-1-6; IEC 60601-1-11

### 3. Preparation and Usage of This Device

Prior to using the device for the first time, we recommend cleaning it as described in the section «Cleaning and Disinfecting».

- Assemble the nebuliser kit (1). Ensure that all parts are complete.
- Fill the nebuliser with the inhalation solution as per your doctor's instructions. Ensure that you do not exceed the maximum level.
- Connect the nebuliser with the air tube (6) to the compressor (2) and plug the power lead (2) into the socket (230V 50 Hz AC).

Ensure that children do not use this device unsupervised; some parts are small enough to be swallowed. Be aware of the risk of strangulation in case this device is supplied with cables or tubes.

Use of this device is not intended as a substitute for a consultation with your doctor.

### 4. Cleaning and Disinfecting

Thoroughly clean all components to remove medication residuals and possible impurities after each treatment. Use a soft and dry cloth with non-abrasive cleaners to clean the compressor.

The device requires no calibration.

No modification to the device is permitted.

Make sure that the internal parts of the device are not in contact with liquids and that the power plug is disconnected.

### 5. Cleaning and disinfecting of the accessories

Follow carefully the cleaning and disinfecting instructions of the accessories as they are very important to the performance of the device and success of the therapy.

Before and after each treatment

Disassemble the nebuliser (5) by turning the top counter-clockwise and remove the medicine container cone. Wash the components of the disassembled nebuliser, the mouthpiece (7) and the noise piece (12) by using tap water, dip in boiling water for 5 minutes. Reassemble the nebuliser components and connect it to the air tube connector, switch the device on and let it work for 10-15 minutes.

Wash masks and air tube with warm water.

Only use cold disinfecting liquids following the manufacturer's instructions.

Do not boil nor autoclave the air tube and masks.

### 5. Maintenance, Care, and Service

Order all spare parts from your dealer or pharmacist, or contact MicroLife-Service (see foreword).

### 6. Malfunctions and Actions to take

The device cannot be switched on

- Ensure the power lead (2) is correctly plugged into the socket.
- Ensure the ON/OFF switch (3) is in the position «I».
- Make sure that the device has been operating within operating limits indicated in this manual (30 min On / 30 min Off).

### 7. Guarantee

This device is covered by a 5 year guarantee from the date of purchase. During this guarantee period, at our discretion, MicroLife will repair or replace the defective product free of charge.

Opening or altering the device invalidates the guarantee. The following items are excluded from the guarantee:

- Transport costs and risks of transport.
- Damage caused by incorrect application or non-compliance with the instructions for use.
- Damage caused by use of adapters, simple or multiple, indicated on adapters and extension cables.
- Damage caused by accident or misuse.
- Packaging/storage material and instructions for use.
- Regular checks and maintenance (calibration).
- Accessories and wearing parts: Nebuliser, masks, mouthpiece, noise piece, tube, filters, nasal washer (optional).

Should guarantee service be required, please contact the dealer from where the product was purchased, or your local MicroLife service. You may contact your local MicroLife service through our website: www.microlife.com/support

Compensation is limited to the value of the product. The guarantee will not be granted if the complete product is returned with the original invoice. Repair or replacement within guarantee does not prolong or renew the guarantee period. The legal claims and rights of consumers are not limited by this guarantee.

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