# beurer

## FM 250 Vital Legs

|   |   |
|---|---|
| <b>DE</b> EMS-Durchblutungsstimulator<br>Gebrauchsanweisung.....2           | <b>IT</b> Stimolatore della circolazione EMS<br>Istruzioni per l'uso .....73  |
| <b>EN</b> EMS circulation stimulator<br>Instructions for use.....20         | <b>TR</b> EMS dolařım uyarıcısı<br>Kullanım kılavuzu.....91                   |
| <b>FR</b> Stimulateur circulatoire EMS<br>Mode d'emploi.....37              | <b>RU</b> Стимулятор кровоснабжения EMS<br>Инструкция по применению ..... 108 |
| <b>ES</b> Estimulador de la circulación EMS<br>Instrucciones de uso .....55 | <b>PL</b> Stymulator krążenia krwi EMS<br>Instrukcja obsługi ..... 126        |

## Inhalt

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Germany

## Contents

|                                      |    |                                      |    |
|--------------------------------------|----|--------------------------------------|----|
| 1. Getting to know your device ..... | 22 | 6. Use .....                         | 30 |
| 2. Intended use.....                 | 22 | 7. Cleaning and storage .....        | 33 |
| 3. Safety notes.....                 | 22 | 8. What if there are problems? ..... | 33 |
| 4. Device description.....           | 28 | 9. Technical data .....              | 35 |
| 5. Initial use .....                 | 29 | 10. Warranty/service.....            | 36 |

**Read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.**

## Dear customer,

Thank you for choosing a product from our range. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage, beauty, baby and air.

With kind regards,  
Your Beurer team

## Included in delivery

Check that the exterior of the delivery cardboard packaging is intact and make sure that all components are present. Before use, ensure that the device and accessories show no visible damage and all packaging material has been removed. If you have any doubts, do not use the device and contact your retailer or the specified Customer Service address.



- 1x EMS circulation stimulator
- 2x cuffs for lower leg
- 1x mains adapter
- 2x connection cables

- 1x remote control
- These instructions for use
- 2x AAA batteries for the remote control
- 4x AA batteries for the EMS circulation stimulator
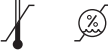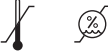
## Signs and symbols

The following symbols appear in the instructions for use and on the type plate

-  **WARNING** Warning notice indicating a risk of injury or damage to health.
-  **IMPORTANT** Safety note indicating possible damage to the device/accessory.
-  **Note** Note on important information.
-  Only use indoors.
-  Application part, type BF
-  The CE labelling certifies that the product complies with the essential requirements of Directive 93/42/EEC on medical devices.
-  Observe the instructions for use
-  Serial number
- IP 21** Protected against foreign objects  $\geq 12.5$  mm and against vertically falling drops of water
- IP 22** Protected against foreign objects  $\geq 12.5$  mm and against vertically falling drops of water with enclosure tilted up to 15 degrees from the vertical.
-  Green dot (Der Grüne Punkt): German dual waste collection system
-  Do not use the EMS circulation stimulator if you have implanted electrical devices (e.g. a pacemaker)
-  Permissible storage temperature and humidity
-  Permissible operating temperature and humidity
-  Disposal in accordance with the Waste Electrical and Electronic Equipment EC Directive – WEEE
-  The device can emit effective output values above 10 mA, averaged over every five-second interval
-  Manufacturer
-  Dispose of packaging in an environmentally friendly manner

## 1. Getting to know your device

The beurer EMS circulation stimulator Vital Legs is an electrostimulation device. The EMS circulation stimulator causes electrical stimulation of the muscle tissue (EMS) in the foot and lower leg area.

The EMS circulation stimulator has been specifically developed for the following purposes:

- Reduce swollen feet and ankles
- Alleviate tired and aching legs
- Improve blood circulation

### How does electrostimulation work?

The principle of electrostimulation devices is based on the imitation of impulses in our bodies that are transferred to nerve and muscle fibres with electrodes via our skin. The electrical impulses are safe and virtually painless. You may merely notice a slight tingling or vibrating sensation. The electrical impulses that are sent into the tissue influence the transmission of stimulation into nerves, nerve centres and muscle groups in the application area.

Electrical muscle stimulation (EMS) is a widespread and generally recognised method, and has been used in sports and rehabilitation medicine for years.

## 2. Intended use

This device is intended only for electrostimulation of human feet and lower legs. The device is only suitable for **private use**, and not for medical or commercial purposes. It cannot replace medical treatment. Do not use the device if one or several of the following warnings applies to you. If you are unsure whether the device is suitable for you, please consult your doctor. Use the device for the purpose it was developed for only and in the manner specified in these instructions for use. Any form of improper use can be dangerous. The manufacturer is not liable for damage resulting from improper or incorrect use.

## 3. Safety notes

Read these instructions for use carefully! Non-observance of the following information may result in personal injury or material damage. The device is not a substitute for medical consultation and treatment. Consult your doctor first if you are experiencing any pain or are suffering from an illness.

### WARNING

- Keep the device and packaging material away from children. The mains adapter cable poses a risk of strangulation and there is a risk of choking due to small parts that can be swallowed.
- This device is not intended for use by persons with physical, sensory (e.g. insensitivity to pain) or mental impairments or those who do not know how to use it due to a lack of experience or knowledge, unless they are supervised by a person responsible for their safety or have received instructions from them on how to use the device.
- Children must not play with the device.
- Cleaning and user maintenance must not be performed by children unless supervised.

**To avoid damage to health, we strongly advise against using the device in the following situations:**

- Do not use the device if you have a pacemaker or other implants, such as an insulin pump or an implanted defibrillator.
- Do not use the device on people who have deep vein thrombosis.
- Do not use the device on people who have arterial or venous thrombosis or thrombophlebitis. There is a residual risk that electrical stimulation of blood vessels at or next to blood clots may lead to an embolism. For this reason, do not use the device on people with thrombosis, even if the condition occurred several years ago.
- Do not use the device on people with metal implants.
- Do not use the device on people who use insulin pumps.
- Do not use the device on people with a high fever (e.g. > 39°C).
- Do not use the device on people with known or acute cardiac arrhythmia or disorders of the heart's impulse and conduction system.
- Do not use the device on people with seizure disorders (e.g. epilepsy).
- Do not use the device during pregnancy.
- Do not use the device on people with cancer.
- Do not use the device near or on cancer wounds.
- To avoid unwanted movement, do not use the device on broken bones.
- Do not use the device after an operation, if strong muscle contractions could affect the healing process.
- Do not use the device on acutely or chronically diseased, injured or irritated skin (e.g. inflamed skin – whether painful or not, reddened skin, rashes, venous inflammation, arterial thrombosis, venous thrombosis, thrombophlebitis, varicose veins, allergies, burns, bruises, swellings, wounds that are still healing or post-operative scars). Only use the device on healthy and clean skin.
- Do not use the device on tissue that is prone to bleeding.
- Do not position the electrodes over the eyes or in orifices.
- Do not use the device in environments with high humidity, such as in the bathroom or when bathing or showering.
- Do not use the device following alcohol consumption.
- Do not use the device if connected to a high-frequency surgical device.
- Do not use the device in the case of acute or chronic diseases of the gastrointestinal tract.
- Do not use the device in the vicinity of electronic monitoring devices (such as heart monitors or electrocardiograph machines).
- Do not use the device in the vicinity of devices that cause unprotected emission of electromagnetic energy.
- Do not use the device near to the heart. This can increase the risk of ventricular fibrillation and induce cardiac arrest.

**Before using the device, consult your doctor if any of the following applies to you:**

- If you have an acute medical condition, in particular if you suspect or have been diagnosed with high blood pressure, a blood coagulation disorder, propensity to thrombo-embolic conditions or in the case of malignant neoplasms
- If you have any skin conditions
- If you have unexplained chronic pain in any part of the body.
- Diabetes
- If you have any sensory impairment that reduces the feeling of pain (e.g. metabolic disorders)
- If you are receiving medical treatment
- In the event of complaints linked to the stimulation treatment
- If you suffer from persistently irritated skin due to long-term stimulation at the same electrode site.

**WARNING**

- Hold the device away from sources of heat and do not use it in close proximity (approx. 1 m) to shortwave or microwave devices (e.g. mobile phones), as doing so can result in unpleasant current peaks.
- Do not expose the device to direct sunlight or high temperatures.
- Protect the device from dust, dirt and humidity.
- Never immerse the device in water or other liquids.
- If the device does not work properly, or you feel unwell or experience pain, stop using it immediately.
- The device is suitable for self-treatment.
- Switch off the device or the respective channel first before removing or relocating the electrodes to prevent unintentional stimulation.
- Do not modify the device.
- Do not modify electrodes (e.g. by cutting them). This increases the current density, which is potentially hazardous (max. recommended output value for the electrodes is 9 mA/cm<sup>2</sup>, an effective current density beyond 2 mA/cm<sup>2</sup> requires increased awareness).
- Do not use the device whilst asleep, driving a vehicle or operating machinery.
- Do not use any sharp objects such as the tip of a pencil or a ballpoint pen to operate the buttons.
- Do not use the device in the immediate vicinity of other electronic devices or between other electronic devices. If this cannot be avoided, before using the device you must check that it functions correctly in the intended location.
- This device is subject to special precautionary measures with regard to EMC and must be installed and commissioned in accordance with these.
- The device has been thoroughly tested and checked to guarantee that it performs and functions correctly.
- Do not use any other high-frequency communication devices or RFID systems in the vicinity of the device.
- Do not use whilst undertaking any activity where an unexpected reaction (e.g. strong muscle contractions even at low intensity) could be dangerous.
- Ensure that no metallic objects can come into contact with the electrode surfaces during stimulation. If you are wearing jewellery or have piercings in the area to be treated, these must be removed before using the device. Failure to do so could result in spot burns.
- Keep the device away from children to prevent potential risks.

- Make sure not to confuse the connection cables and their contacts with your headphones or other devices. Do not connect the cuffs to other devices.
- Do not use the device whilst using other devices that transmit electrical impulses into your body.
- Do not use the device in the vicinity of highly flammable substances, gases or explosives.
- Do not use rechargeable batteries. Always use the same battery types.
- Do not apply greasy creams or lotions to the skin before using the device. This may cause unpleasant current peaks. You can use water or electrode gel to improve the contact between your feet and the electrodes.

### **Precautions**

- Use the device on humans only.
- Use the device for the purpose it was developed for only and in the manner specified in these instructions for use. Any form of improper use can be dangerous.
- The device is intended for external use only.
- The long-term effects of chronic electrostimulation are unknown.
- Electrical muscle stimulation devices have no healing effect.
- Electrostimulation or the electrodes may cause skin irritation in sensitive persons.
- Only use the device with the lower-leg cuffs included in delivery. Do not use the device with cuffs or electrodes from other manufacturers.

### **Possible side effects**

- The possible side effects are: skin irritation, inflammation and burns beneath the electrodes.

### **Electric shock (for use with mains part, without batteries)**

#### **WARNING**

As with every electrical device, use the EMS circulation stimulator carefully and cautiously in order to avoid hazards due to electric shock.

The device should therefore

- only be used with the supplied mains part and at the mains voltage specified on the mains part,
- never be used if it or its accessories show visible signs of damage,
- not be used during a storm.

In the event of defects or operational faults, switch the device off immediately and disconnect it from the mains socket. Do not pull on the mains cable or on the device to disconnect the mains part from the socket. Never hold or carry the device by the mains cable. Keep the mains cable away from hot surfaces. Do not crush, bend or twist the mains cable. Do not insert needles or other sharp objects into the device.

Ensure that the device, the mains part and the mains cable do not come into contact with water, steam or other liquids. Only use the device in dry rooms (e.g. never in the bath, sauna).

Never reach for a device that is submerged in water. Unplug the mains part from the socket immediately. Do not use the device if it or its accessories are visibly damaged.

Do not drop the device or subject it to any impacts.

## Damage

### **IMPORTANT**

- Only specialist personnel may perform repairs on electrical devices. Improper repairs may subject users to considerable danger. For repairs, please contact Customer Services or an authorised retailer.
- If the device is damaged, do not use it and contact your retailer or the specified Customer Services address.
- To ensure that the device functions effectively, do not drop it or dismantle it.
- Check the device for signs of wear and tear or damage. If there are such signs of wear and tear or damage or if the device was used improperly, it must be returned to the manufacturer or retailer before further use.
- Switch the device off immediately if it is faulty or not working properly.
- Never attempt to open and/or repair the device yourself. Repairs may only be carried out by Customer Services or authorised retailers. Failure to comply with this instruction will void the warranty.
- The manufacturer is not liable for damage resulting from improper or careless use.

## Risk of fire

### **WARNING**

There is a risk of fire if the device is used improperly or these instructions for use are ignored.

The device should therefore

- never be used underneath a cover, such as blanket, pillow, etc.
- never be used near petrol or other highly flammable substances.

## Handling

### **IMPORTANT**

The device must be switched off and disconnected after every use and before cleaning.

## **Disposal**

### **IMPORTANT**

For environmental reasons, do not dispose of the device in the household waste at the end of its service life.

Dispose of the device at a suitable local collection or recycling point in your country.

Dispose of the device in accordance with the Waste Electrical and Electronic Equipment EC Directive – WEEE. If you have any questions, please contact the local authorities responsible for waste disposal. The empty, completely flat batteries must be disposed of through specially designated collection boxes, recycling points or electronics retailers. You are legally required to dispose of the batteries.

The codes below are printed on batteries containing harmful substances:

Pb = Battery contains lead,

Cd = Battery contains cadmium,




Hg = Battery contains mercury.

### **Notes on handling batteries**

- If your skin or eyes come into contact with battery fluid, rinse the affected area with water and seek medical assistance.
- ⚠ Choking hazard! Small children may swallow and choke on batteries. Store batteries out of the reach of small children.
- Observe the plus (+) and minus (-) polarity signs.
- If a battery has leaked, put on protective gloves and clean the battery compartment with a dry cloth.
- Protect batteries from excessive heat.
- ⚠ Risk of explosion! Do not throw batteries into a fire.
- Do not charge or short-circuit batteries.
- If the device will not be used for some time, take the batteries out of the battery compartment.
- Use identical or equivalent battery types only.
- Always replace all batteries at the same time.
- Do not use rechargeable batteries.
- Do not disassemble, open or crush the batteries.

## 4. Device description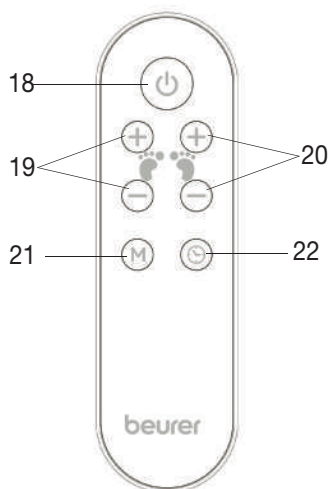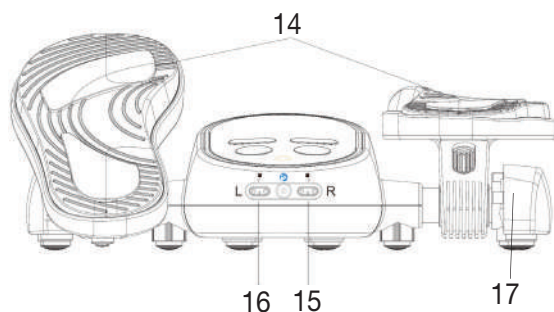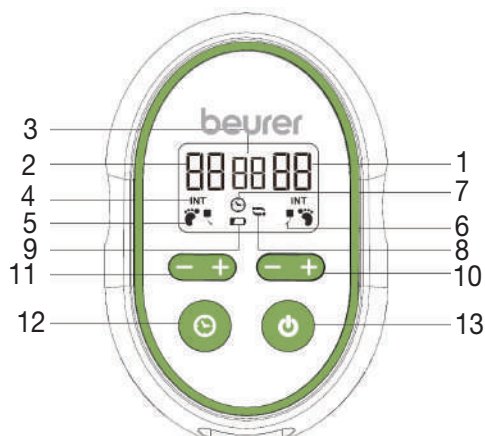

1. Right intensity level (foot/lower leg)
2. Left intensity level (foot/lower leg)
3. Remaining usage time in minutes
4. Intensity symbol
5. Foot stimulation mode
6. Foot and lower leg stimulation mode
7. Treatment time symbol
8. Alternating stimulation mode symbol
9. Battery status symbol
10. Increase/decrease intensity level (right foot/lower leg)
11. Increase/decrease intensity level (left foot/lower leg)
12. Set treatment time
13. ON/OFF button
14. Foot electrode surface
15. Connection socket for the right lower-leg cuff
16. Connection socket for the left lower-leg cuff
17. Side supports that pull out for the step function

18. ON/OFF button
19. Intensity level for left leg
20. Intensity level for right leg
21. Mode button for selecting one of the 15 pulse wave variations
22. Timer button

## 5. Initial use


### Note

You can use the device either with the supplied mains adaptor or with batteries. We recommend using the mains adaptor for reasons of sustainability.

### Inserting the batteries

To operate the device with batteries, you must first insert the batteries into the EMS circulation stimulator. Proceed as follows.


1. Open the battery compartment lid on the bottom of the EMS circulation stimulator.
2. Insert four AA (1.5 V) batteries into the battery compartment. Make sure that the batteries are inserted the right way round (+/-), as shown in the battery compartment.
3. Push the battery compartment closed until you hear and feel it click into place.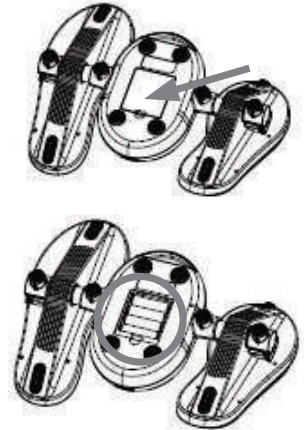

### Inserting the batteries in the remote control

To use the remote control, you must first insert batteries in the remote control. Proceed as follows.

1. Open the battery compartment lid on the bottom of the remote control by using a pointed object to push the lock back and release the cover.
2. Insert two AAA (1.5 V) batteries into the battery compartment. Make sure that the batteries are inserted the right way round (+/-), as shown in the battery compartment.
3. Push the battery compartment of the remote control closed until you hear and feel it click into place.

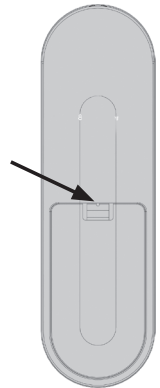### Connecting the mains adapter

To operate the device with the supplied mains adapter, you need to connect the mains adapter to the EMS circulation stimulator. Proceed as follows:

1. Connect the cable of the mains adapter to the EMS circulation stimulator's connection socket.
2. Insert the mains adapter into a suitable socket.
3. If you want to use the device just with the mains adapter, do not insert any batteries in the device.

**WARNING** Trip hazard! Ensure that the mains cable does not pose a trip hazard.

## 6. Use

### Note

- As well as your feet, you can also use the EMS circulation stimulator to stimulate your lower legs.
- Do not use the EMS circulation stimulator for longer than 90 minutes a day. If it is used for longer periods, the overstimulation may cause tension instead of aiding relaxation.
- The EMS circulation stimulator switches off automatically once the set usage time has ended.

### WARNING

- Only use the EMS circulation stimulator when you are sitting down. Never use the EMS circulation stimulator when standing up.


### 6.1 Foot stimulation

To start foot stimulation, proceed as follows:

|   |   |
|---|---|
| <p><b>Step 1: Place the device on the floor</b></p> <ul style="list-style-type: none"><li>• Place the EMS circulation stimulator on an even, hard surface. Position a chair in front of the EMS circulation stimulator so that you can reach the device comfortably with your feet when sitting in the chair.</li></ul> <p> <b>WARNING</b> Trip hazard! Never use the EMS circulation stimulator when standing up.</p>   |    |
| <p><b>Step 2: Place both feet on the device</b></p> <ul style="list-style-type: none"><li>• Place your bare feet on the foot areas of the EMS circulation stimulator. Make sure that the arch of your foot sits on the central raised section. It does not matter if your feet are bigger or smaller than the foot areas of the EMS circulation stimulator.</li></ul> <p> <b>IMPORTANT</b> Make sure that your knees form an angle of around 90 degrees when you are sitting.</p> <p><b>Step function</b></p> <p>The EMS circulation stimulator also features a step function. You can use the step function to tilt your feet forwards and backwards during electrostimulation. The additional movement of your ankle boosts circulation and this can reduce swelling. To achieve the best possible results, we recommend that you perform electrostimulation in combination with the step function.</p> <ul style="list-style-type: none"><li>• To use the step function, pull out the supports on the sides of the device. You can now tilt the foot areas forwards and backwards.</li><li>• If you find the step function uncomfortable or want to stop the function, push the side supports into the device again until you feel them click into place. You can lock the foot areas into place either in a horizontal position or tilted forwards.</li></ul> |  |


### Step 3: Switch on the device

- To switch on the EMS circulation stimulator, press the 0 ON/OFF button on the device or on the remote control. Initially, the intensity level 00 is set on the left and the right side.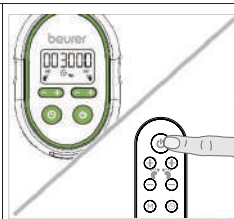


### Step 4: Set the usage time

- Use the ⏱ button on the device or remote control to select the usage time you want. You can set a usage time of between 5 and 90 minutes (in 5-minute increments).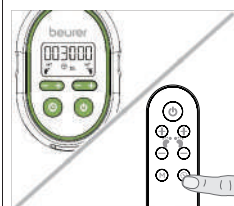

### Step 5: Start electrostimulation / set the intensity level

- Use the +/- buttons on the device or remote control to select the intensity level you want for the left foot and the right foot. You can set an intensity level of between 1 and 99. Press the +/- button briefly to set the intensity level, step by step. Press and hold the +/- button to set the desired intensity level more quickly.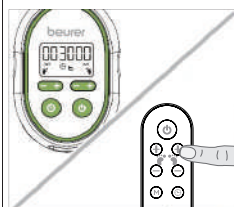

**!** **IMPORTANT** Start with a low intensity level at first so that you get used to the device gradually.

- If it is hard to feel the electrostimulation, you can moisten the soles of your feet slightly with water. This provides better conduction of the electricity. Do not use cream or lotion; only use water or a water-based electrode gel.

### **i** Note

- During electrostimulation, 15 different pulse wave variations run in a loop for one minute each. To set one of the pulse wave variations permanently, press the **M** button on the remote control. If you want the pulse wave variations to be varied again, press the **M** button again.


### Detection for loss of body contact

- If you take your feet off the foot areas of the EMS circulation stimulator for less than 5 seconds, the device emits three short beeps. As soon as you put your feet back on the foot areas, the intensity level that you previously selected is set again.
- If you take your feet off the foot areas of the EMS circulation stimulator for more than 5 seconds, the intensity level will reset to 0. To continue the stimulation, increase the intensity level again on the EMS circulation stimulator or on the remote control.

### Step 6: Switch off the device

- To switch off the EMS circulation stimulator, briefly press the 0 ON/OFF button on the device or on the remote control.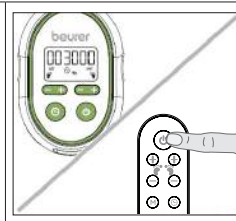

## 6.2 Lower leg stimulation

If you want to stimulate your lower legs at the same time as your feet, proceed as follows:

|   |  |
|---|--|
| <p><b>Step 1: Moisten the cuffs</b></p> <ul style="list-style-type: none"><li>• Remove the two cuffs from the packaging. Moisten the two cuffs with water or a water-based electrode gel. This provides better conduction of the electricity. Do not use cream or lotion; only use water or a water-based electrode gel.</li></ul>  |  |
| <p><b>Step 2: Connect the connection cables to the cuff</b></p> <ul style="list-style-type: none"><li>• Connect the connection cables to the press stud on the cuff as shown. Ensure that the connection cable is firmly connected to the cuff.</li></ul>   |  |
| <p><b>Step 3: Connect the connection cables to the device</b></p> <p> <b>WARNING</b></p> <p>Ensure that the device is switched off.</p> <ul style="list-style-type: none"><li>• Insert the connection cables into the L and R connection sockets on the front of the device.</li></ul>  |  |
| <p><b>Step 4: Position the cuffs on your lower legs</b></p> <ul style="list-style-type: none"><li>• Place the cuffs on your lower legs as shown. The electrode should be positioned on your calf and may reach as far as your shin, depending on the circumference of your leg.</li><li>• Make sure that the cuffs are fastened tightly, but still feel comfortable. Position the connection cables so that you can easily move your feet back and forth on the foot areas of the device.</li></ul> |  |
| <p><b>Step 5: Start lower leg stimulation</b></p> <ul style="list-style-type: none"><li>• To start lower leg stimulation, follow steps 1 to 6 from section "6.1 Foot stimulation".</li></ul>  |  |

## 7. Cleaning and storage

### Cleaning

#### WARNING

- Always unplug the mains part from the socket and disconnect it from the device before carrying out any cleaning.
- Always remove the batteries before cleaning the EMS circulation stimulator.
- Only clean the appliance in the manner specified. Fluid must never be allowed to enter into the device or accessories.
- Clean the electrode surfaces on the EMS circulation stimulator and the cuffs with a cloth or damp sponge after each use. Do not use any solvent-based cleaning products.
- Leave the EMS circulation stimulator to dry at room temperature, away from heat and sunlight.
- Do not use the EMS circulation stimulator again until it is completely dry.

### Storage

If you do not plan to use the EMS circulation stimulator for a long period of time, we recommend that you store it in the original packing in a dry environment, ensuring it is not weighed down by additional items. Store the device away from children, pets and pests. Do not place any objects on top of the EMS circulation stimulator. Remove the batteries from the EMS circulation stimulator and remote control. Make sure that the storage conditions are met in the storage location.

## 8. What if there are problems?

| Problem   | Cause   | Solution   |
|---|---|--|
| <b>The EMS circulation stimulator will not switch on</b>                                      | The batteries are flat.                           | Replace the batteries in the EMS circulation stimulator.   |
|   | The batteries are not inserted correctly.         | Ensure that the batteries are inserted correctly. Observe the plus (+) and minus (-) polarity signs. |
|   | The EMS circulation stimulator is not plugged in. | Connect the mains adapter to the EMS circulation stimulator and a suitable socket.                   |
| <b>The remote control is not responding.</b>  | The batteries are flat.                           | Replace the batteries in the remote control.   |
|   | The batteries are not inserted correctly.         | Ensure that the batteries are inserted correctly. Observe the plus (+) and minus (-) polarity signs. |
| <b>The EMS circulation stimulator is switched on but I cannot feel any electrostimulation</b> | Your feet are too dry.                            | Moisten the bottom of your feet. If necessary, increase the intensity level.                         |
| <b>The electrostimulation is too weak.</b>  | The set intensity level is too low.               | Increase the intensity level.  |

|  |   |  |
|--|---|--|
| <b>The electrostimulation is unpleasant.</b>           | The set intensity level is too high.  | Decrease the intensity level.  |
|  | The EMS circulation stimulator is not being used in accordance with these instructions for use. | Read the instructions for use before you start using the device.   |
| <b>My calves and feet hurt after using the device.</b> | The set intensity level is too high. The muscles are overstimulated.                            | Decrease the intensity level. Take adequate breaks between each use of the device so that your muscles can recover.  |
| <b>Use of the device was interrupted.</b>              | The batteries are flat.   | Replace the batteries in the EMS circulation stimulator.   |
|  | The EMS circulation stimulator switches off automatically once the set usage time has ended     | Press the ON/OFF button  to switch on the EMS circulation stimulator again. |

## 9. Technical data

|  |   |
|--|---|
| Type   | FM 250  |
| Output waveform                              | Biphasic rectangular pulses   |
| Pulse frequency                              | 10 Hz~36 Hz   |
| Pulse length                                 | 250 µs  |
| Output voltage                               | 0 ~ 80 Vpp (at 500 Ω)   |
| Voltage supply<br>EMS circulation stimulator | Mains adapter input: 100–240 V~; 50/60 Hz 0.2 A;<br>Output: DC 6.0 V; 300 mA;<br>or<br>DC 6.0 V; 4 x AA batteries |
| Voltage supply<br>Remote control             | DC 3.0 V; 2 x AAA batteries   |
| Battery life                                 | 1 week (for daily use of 30 minutes)  |
| Channels                                     | 2 channels  |
| Programs                                     | 1 program with 15 repeating pulse wave variations   |
| Treatment time                               | 5–90 minutes (in 5-minute increments)   |
| Intensity levels                             | 1-99  |
| Operating conditions                         | 5°C – 40°C (41°F – 104°F) at a relative humidity of 30 – 75%;<br>700 hPa – 1060 hPa                               |
| Storage and transportation<br>conditions     | -10 °C – 55 °C (14 °F – 131 °F) at a relative humidity of 10<br>– 90 %; 700 hPa – 1060 hPa                        |
| Dimensions                                   | EMS circulation stimulator:<br>approx. 283.3 x 372.2 x 111.4 mm   |
| Weight                                       | 1,500 g (only EMS circulation stimulator, without batteries)  |

The serial number is located on the device or in the battery compartment.

Battery life: With included batteries at 500Ω approx. 8 applications (30 minutes each).

Note: If the device is not used according to the instructions specified, perfect functionality cannot be guaranteed! We reserve the right to make technical changes to improve and develop the product.

This device complies with European standards EN60601-1 and EN60601-1-2 (in accordance with CISPR 11, IEC 61000-3-2, IEC 61000-3-3, IEC 61000-4-2, IEC 61000-4-3, IEC 61000-4-4, IEC 61000-4-5, IEC 61000-4-6, IEC 61000-4-8, IEC 61000-4-11) and is subject to special precautionary measures with regard to electromagnetic compatibility. Please note that portable and mobile HF communication systems may interfere with this device. For more details, please contact our Customer Services at the address indicated.

This device meets the requirements of European Directive 93/42/EEC for medical devices, as well as those of the Medizinproduktegesetz (German Medical Devices Act). For this device, a functional test and instruction in accordance with Section 10 of the Medical Devices Operator Ordinance (MPBetreibV) is not required.

## Notes on electromagnetic compatibility

### **WARNING**

- The device is suitable for use in all environments listed in these instructions for use, including domestic environments.
- The use of the device may be limited in the presence of electromagnetic disturbances. This could result in issues such as error messages or the failure of the display/device.
- Avoid using this device directly next to other devices or stacked on top of other devices, as this could lead to faulty operation. If, however, it is necessary to use the device in the manner stated, this device as well as the other devices must be monitored to ensure they are working properly.
- The use of accessories other than those specified or provided by the manufacturer of this device can lead to an increase in electromagnetic emissions or a decrease in the device's electromagnetic immunity; this can result in faulty operation.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should not be used closer than 30 cm to any part of the device, including cables specified by manufacturer. Otherwise, degradation of the performance of this equipment could result.
- Failure to comply with the above can impair the performance of the device.

## **10. Warranty/service**

In case of a claim under the warranty please contact your local dealer or the local representation which is mentioned in the list "service international".

In case of returning the unit please add a copy of your receipt and a short report of the defect.

The following warranty terms shall apply:

1. The warranty period for BEURER products is either 3 years or- if longer- the country specific warranty period from date of Purchase. In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Repairs (complete unit or parts of the unit) do not extend the warranty period.
3. The warranty shall not be valid for damages because of
  - a. improper treatment, e.g. nonobservance of the user instructions.
  - b. repairs or tampering by the customer or unauthorised third parties.
  - c. transport from the manufacturer to the consumer or during transport to the service centre.
  - d. The warranty shall not be valid for accessories which are subject to normal wear and tear (cuff, batteries etc.).
4. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

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Beurer GmbH • Söflinger Straße 218 • 89077 Ulm, Germany