

IS YESON

NINGBO YINZHOU YESON MEDICAL DEVICE CO.,LTD.

No. 1 Huanzhen Rd., Hengjie Town, Yinzhou, Ningbo, China 315181

Tel.: +86-574-8842 5999

Fax: +86-574-8828 3099

E-mail: yeson@yeson-medicine.com

www.yeson-medicine.com



IS YESON

YESON

International Sterilization Solutions





Yeson

Yeson Medical, founded in year 2005, is a high-tech enterprise in China specialized in the R&D, production and marketing of medical sterilizers/autoclaves. We have a young ambitious management team that has professional knowledge of medical devices, strict quality control system, good skills of global marketing. We are aiming at being a world-class sterilization solutions supplier. We are now exporting autoclaves to more than 50 countries with top quality, competitive prices, good performance. We hope our products can help global medical clinics, hospitals to prevent virus infection, and save lives.

Our products include: Class B autoclaves, Class N autoclaves, water distillers, washing machines, sterilization sealer, packing strap etc. Our products are CE 0197 certified.

R&D: We regard R&D as the basis of company's future. We have a most professional R&D team in China, and co-operate with Chinese medical universities to ensure our products will be leading in the global market.

Production: We adopt most strict quality control system during production. We use professional testing devices and testing tools to monitor products' quality. Every part, assembly, autoclaves are 100% inspected before delivery.

Yeson aims to always act with uncompromising integrity, whether dealing with our suppliers, partners, employees, customers, or society more broadly. Our shared values guide us to behave ethically and in compliance with the law, and to respect the rights of all our shareholders.

1. Be honest during communication in company, with partners, customers & suppliers.
2. We devote to supplying our customer top quality products and service.
3. Be responsible for our words and actions.
4. Care our employees and consumers.
5. Obey global laws and regulations.
6. Be fair to our employees, shareholders, partners, customers and suppliers.
7. Respect and listen to our employees, shareholders, partners, customers and suppliers.

World-class Target

We focus on global infection control and aim to be a world-class sterilization solutions supplier.

World-class Design

We deeply understand the soul of autoclaves and devote to designing world-class autoclaves through our professional knowledge & serious attitude.

First-class R&D Team

In 7 years, we've registered more than 20 patents for autoclaves. We have a first-class R&D team with deep understanding & professional skills of medical industry & medical devices. We focus on the future and develop totally new autoclaves.


Professional



Benevolent. Excellent.



More Complete Sterilization
Stability of T/P Curve Twice EN13060 Standard

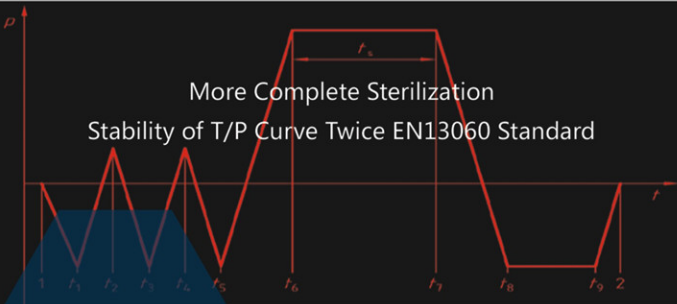

Complete

Complete Sterilization Guaranteed

Yeson knows well that complete sterilization is the only value for an autoclave. We have the unique design to keep the sterilization temperature stable, and the stability level is higher than CE standard. Complete sterilization is guaranteed by YESON autoclaves.

Adjustable Sterilization Time


For special loads, users can adjust sterilization time by themselves to ensure the loads can be completely sterilized.

Adjustable Vacuum Times

For special hollow loads, users can adjust vacuum times accordingly to ensure they will be completely sterilized.

Quick

Our quickest sterilization takes only 12 Min. (including dry time). Your cost and time are saved.

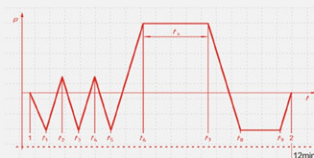

Quick



Fault Self-checking System Makes Maintenance So Easy



Easy

First-class Safety Design

Electrical lock & mechanical lock are both equipped in Yeson autoclaves, to ensure the door will not be opened during sterilization. Operator's safety is guaranteed.

Dual Core Control System

This system can gather and process real-time data, monitoring the sterilization program and automatically adjusting the processing parameters, to ensure the safety of operators and autoclaves.

Power-off Protection

If the power is off during working, the pressure-release valve will be started automatically to release the vapour from the chamber, you can take out sterilization loads safely and timely. Operator's safety is guaranteed.


Safe

Easy Human Machine Interface

What you need to do is just to choose a sterilization program and start it. No difficulty, no training. Simple and easy.

Easy Maintenance

Yeson has a unique fault-self-checking system. It's easy & quick to find the part with problems. No headache in maintenance any more.

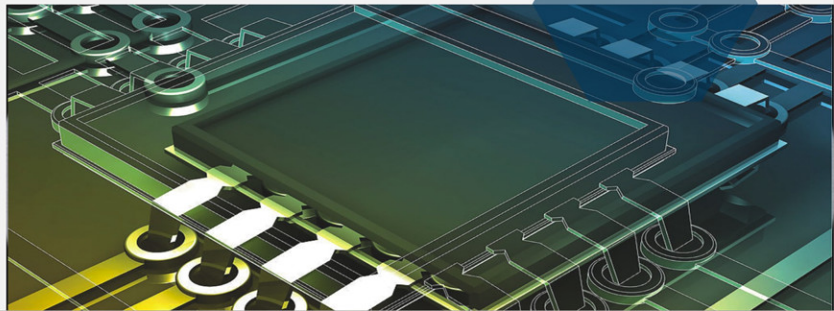

$\geq 95\%$

Qualification Rate in 2-Year Warranty

High-quality

First-class Quality Control System

With a global vision & world-class quality standards, we have established a first-class quality control system to supply the world most excellent autoclaves.

2-Year Warranty

We provide our customers 2-year warranty for each autoclave, and do our best to solve the problems during your after-sale service and ensure your 100% satisfaction.

Energy Saving

Our unique energy saving system makes it possible to energy/water saving and environment protection.

Your medical cost will be saved accordingly.

Filtered Water Can Also Be Used

Not only distilled water, but also filtered water can be used in our new autoclaves.

Your cost on water will be saved.

Save





Class B

The Classic

Professional Design

LED display, easy operation, stainless steel water tank, high density bacteria filter.

Leading Technology

Automatic sterilization, automatic drying, keep-temperature function, adjustable sterilization time, accurate temperature control, safety protection design, monitoring system.

Program	121°C Solid	121°C Universal	134°C Solid	134°C Universal	Cotton	BD/Helix Test	Vacuum Test
Vacuum times	1	3	1	3	3	3	3
Temperature (°C)	121	121	134	134	134	134	/
Pressure (KPA)	110	110	210	210	210	210	-80
Time (Min.)	20	20	4	4	4	3.5	15



Class B

The Competitive

Professional Design

Filtered water can also be used, automatic water filling, automatic water-release, LED display, easy operation, stainless steel water tanks, high density bacteria filter.

Leading Technology

Steam generated inside chamber, automatic sterilization & dry, keep-temperature function adjustable sterilization time, accurate temperature control, safety protection design, easy maintenance.

Program	121°C Solid	121°C Universal	134°C Solid	134°C Universal	Cotton	BD/Helix Test	Vacuum Test
Vacuum times	1	3	1	3	3	3	3
Temperature (°C)	121	121	134	134	134	134	/
Pressure (KPA)	110	110	210	210	210	210	-80
Time (Min.)	20	20	4	4	4	3.5	15



YS-8L

Capacity	8L
Chamber Size	φ17*32cm
Display	LED
Lock	Electrical/Mechanical
Product Size	53.5*36.5*32cm
G.W./N.W.	40/34 KGS
Power	220V/50HZ 1.7KW
Output	RS232
Printer	thermal/ stylus

YS-12L

Capacity	12L
Chamber Size	φ20*36cm
Display	LED
Lock	Electrical/Mechanical
Product Size	66*44.5*39cm
G.W./N.W.	53/45 KGS
Power	220V/50HZ 1.7KW
Output	RS232
Printer	thermal/ stylus

YS-18L

Capacity	18L
Chamber Size	φ25*35cm
Display	LED
Lock	Electrical/Mechanical
Product Size	66*44.5*39cm
G.W./N.W.	56/48 KGS
Power	220V/50HZ 1.7KW
Output	RS232
Printer	thermal/ stylus

YS-22L

Capacity	22L
Chamber Size	φ25*45cm
Display	LED
Lock	Electrical/Mechanical
Product Size	70*44.5*39cm
G.W./N.W.	58/50 KGS
Power	220V/50HZ 1.7KW
Output	RS232
Printer	thermal/ stylus

YS-8L-II

Capacity	8L
Chamber Size	φ17*32cm
Display	LED
Lock	Electrical/Mechanical
Product Size	53.5*36.5*32cm
G.W./N.W.	34/28 KGS
Power	220V/50HZ 1.7KW
Output	RS232
Printer	thermal/ stylus

YS-12L-II

Capacity	12L
Chamber Size	φ20*36cm
Display	LED
Lock	Electrical/Mechanical
Product Size	66*44.5*39cm
G.W./N.W.	46/37 KGS
Power	220V/50HZ 1.7KW
Output	RS232
Printer	thermal/ stylus

YS-18L-II

Capacity	18L
Chamber Size	φ25*35cm
Display	LED
Lock	Electrical/Mechanical
Product Size	66*44.5*39cm
G.W./N.W.	49/40 KGS
Power	220V/50HZ 1.7KW
Output	RS232
Printer	thermal/ stylus

YS-22L-II

Capacity	22L
Chamber Size	φ25*45cm
Display	LED
Lock	Electrical/Mechanical
Product Size	70*44.5*39cm
G.W./N.W.	52/43 KGS
Power	220V/50HZ 1.7KW
Output	RS232
Printer	thermal/ stylus

